


■ How to care for Torque Ratchet

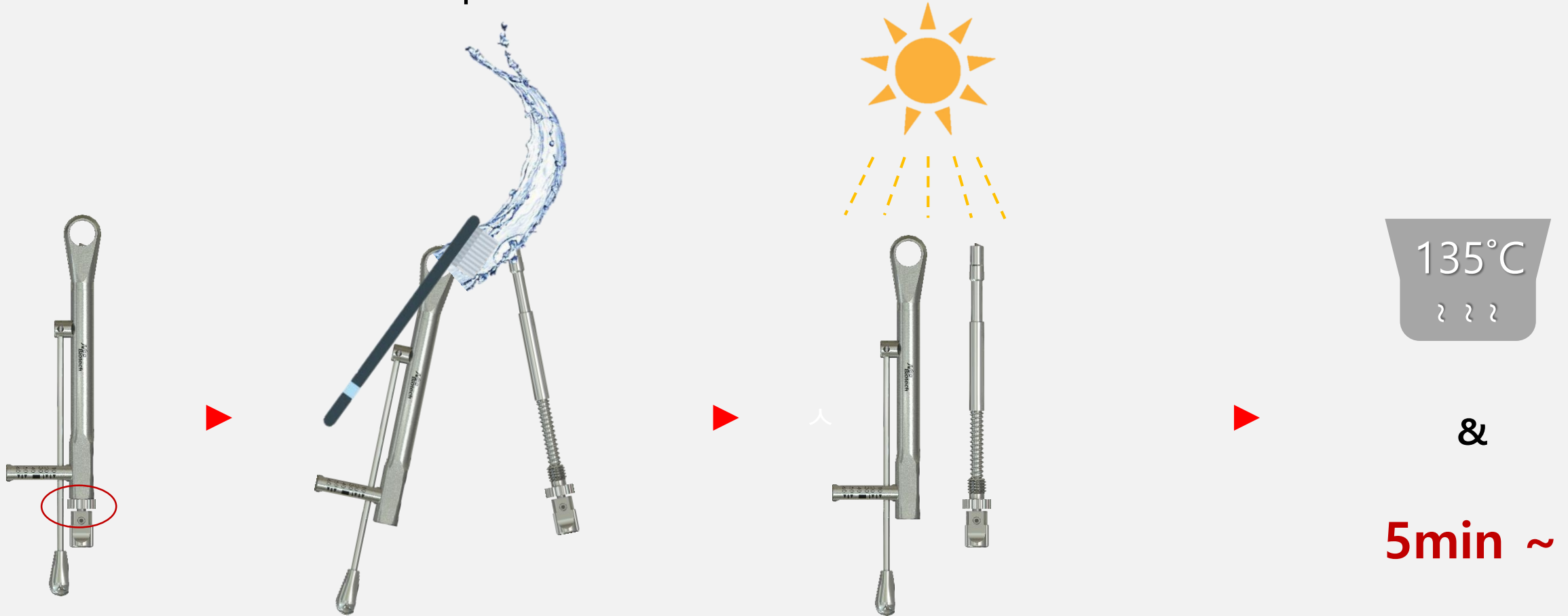

- ① After using the Torque Ratchet, turn the wheel counter-clockwise to separate it.
- ② Using a brush, wash the torque ratchet with running water, remove any residual blood or foreign objects. (Use a syringe or pipe cleaner for areas difficult to wash)
- ③ Dry to remove moisture, sterilize at 135°C or higher degrees in an autoclave.
- ④ Store as separated, and reassemble for surgical use.

Recommended Time Set (Sterilization / Dry)

✓ Pre-vacuum : 5min / 20min


✓ Gravity : 30min / 40min

2. Torque Ratchet Storage Location

1) Neo Master Kit _ Middle tray



2) Neo Surgical Kit / IS Full Kit ver.6 _ Bottom tray

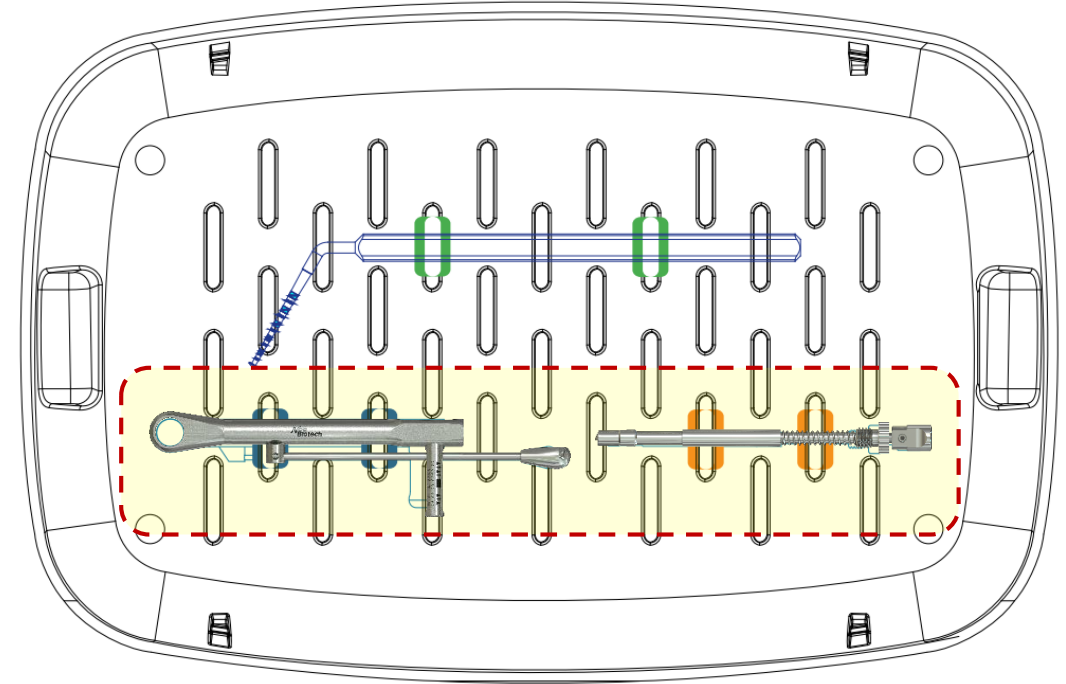

⇒ Torque Ratchet must be separated and stored after cleaning and sterilization.