


Neo Master Kit – Drilling Sequence


Drilling Speed & Torque

- Guide Point Drill, Initial Drill, Twist Drill : 1,200rpm / 35~45Ncm
Cortical Tap / Profile Tap : 50rpm / 50Ncm
Cortical Drill : 1,200rpm / 35~45Ncm (Conventional Loading case)


- IS-III active Fixture $\varnothing 3.5$ X 10mm (D1/D2 bone)

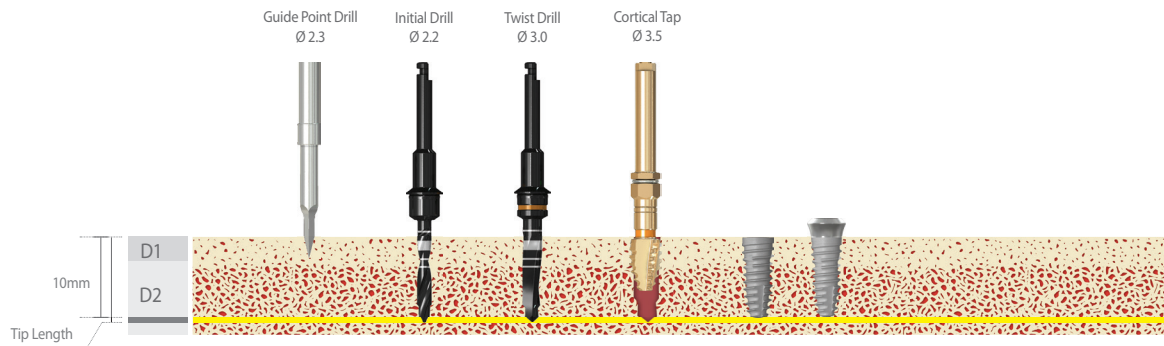

- IS-III active Fixture $\varnothing 4.0$ X 10mm (D1/D2 bone)

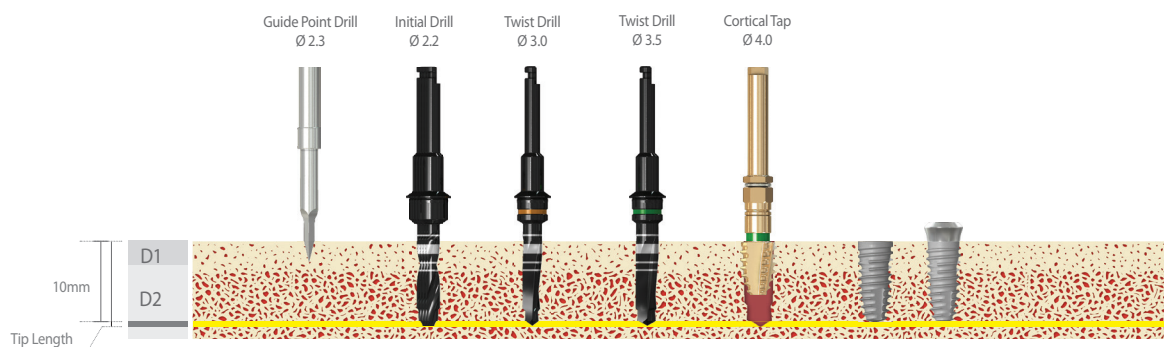

- IS-III active Fixture $\varnothing 4.5$ X 10mm (D1/D2 bone)

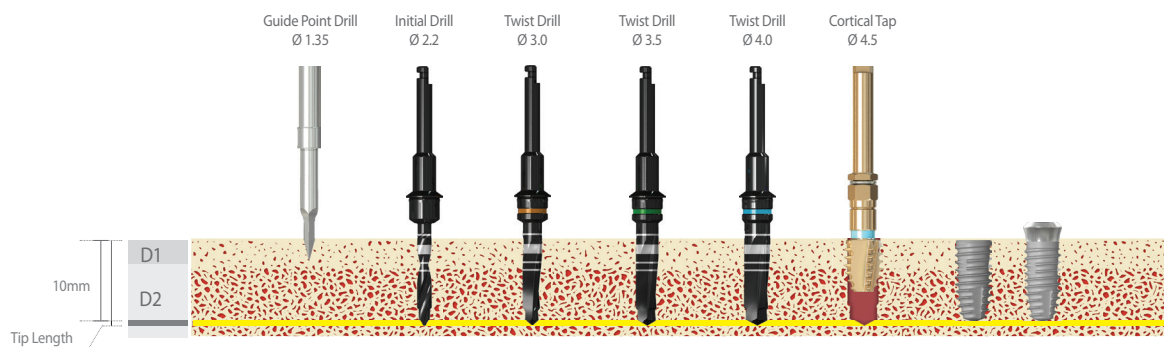

- IS-III active Fixture $\varnothing 5.0$ X 10mm (D1/D2 bone)

